

IVD

PRODUCT INFORMATION

REF 36061003

Negative Control for ECP



Intended purpose

The negative control is only suitable for checking the accuracy of the Omega Diagnostics ECP enzyme immunoassay. The instructions for the use of the test kit must be observed.

Contents

The vial contains 1.3 ml of negative control serum. It contains 1% goat serum.
Preservation agent: 0.09% sodium azide.

Control value

< 2.0 ng/ml

The stated value was determined with the Omega Diagnostics ECP enzyme immunoassay. The value is below the clinical cut-off value and can therefore be used as a negative control serum.

Preparation of the negative control

The control serum must be diluted in the ratio 1+4 with the diluent, i.e. 50 µl of control serum must be diluted with 200 µl diluent solution.

Storage and shelf life


At 2 to 8 °C the serum can be stored until the printed use-by date.

Attention: Do not mix up the screw caps!

Warnings and precautionary measures

The serum was tested by FDA approved methods and was found to be non-reactive for hepatitis-B surface antigen (HB_s Ag), antibodies to hepatitis-C (HCV) and antibodies to HIV-1/HIV-2.
However, all material of human origin must be considered to be potentially infectious. Therefore, handle the serum with the same precautions as used for patient specimens.

Manufacturer

 **Omega Diagnostics GmbH**
Herrengarten 1
D-21465 Reinbek – Germany
☎ ++49-40-636654-100
Fax++49-40-636654-111